
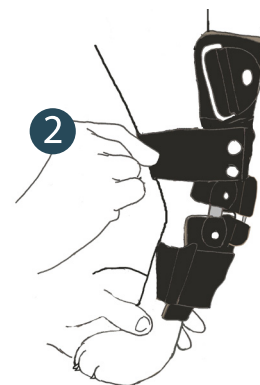


Use an allen key to adjust flexion stop in the carpal joint by turning the anterior screw. Set the hyperextension stop screws at desired flexibility.

At delivery the orthosis is locked in desired angle. When you wish to use the orthosis but without the motion-limiter, loosen screws and metal plate for motion-limiting. Then remove them.

Fitting template

1. Place the leg in the orthosis and push it inwards until stop. Make sure it fits correctly and is properly aligned. The widest part of the carpal joint should be aligned with the widest part of the orthosis at the joint.
2. Begin with tightening the middle strap, alternatively if there is no middle strap, the lower strap. The straps should be tightened so that they firmly hold the orthosis to the leg but not as tight as to where it stops blood circulation. A fingertip should just fit under the strap.

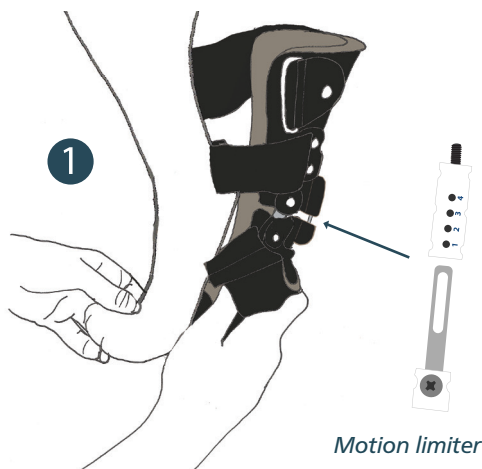



*This applies to all straps.

3. Cut the pads so that they are inside the edge when the strap is tightened. When the straps are well fitted, grip the orthosis with one hand and grip the leg proximal. Feel if there is movement and the leg moves inside the orthosis. The soft tissue should move but the bones should be firmly in place in the orthosis.

4. To assess how the orthosis affects the gait; the orthosis should allow movement without external or internal rotation of the orthosis or leg.
5. Make sure that the leg does not abduct / adduct when walking.

The orthosis can be used around-the-clock if no problems occur. Examine the leg 2-4 times / day for swelling of the toes, as well as for heat gain and chafing.

Contact us if exuding wounds, odours or other problems occur. It is always easier to treat complications that are noticed early, so let them know if there are skin irritations, preferably before it has become a wound or drainage of fluid.

The orthosis always affects the gait. Does it seem uncomfortable for the patient? If the animal licks, bites or is not using the leg, please contact us. Be aware of changing behavior, for example, if the mood is not as it used to, or if the animal does not want to eat. Contact us if you have questions!

Clean the orthosis with soap and water. Keep velcro clean for best adhesion.

Inform the pet owners about Scandi Orthopedic AB. They shall contact us for questions regarding the use of the orthosis and for problem solving.

Tel. 031-21 41 28

